

MEGADYNE™ MEGA SOFT™
Reusable Patient Return Electrode

Cleaning and Care

Below is a reinforcement for key instructions for use of the MEGADYNE™ Patient Return Electrode Pads. Please reference the IFU for complete instructions. Any additional questions, please reach out to your Ethicon Sales Representative or visit our webpage <https://www.jnjmedtech.com/en-EMEA/support/customer-service/ethicon>.



DO

Step 1: CLEAN

- ✓ Clean/disinfect the Patient Return Electrode Pad and cable (e.g., a mild bleach solution (10:1)).

Step 2: RINSE

- ✓ Thoroughly rinse the Patient Return Electrode Pad with clear water to remove any residue from cleaning solutions.
- ✓ Running water is not required for rinsing; may be wiped with a wet cloth.

Step 3: DRY

- ✓ Completely dry both sides of the Patient Return Electrode Pad after each cleaning.
- ✓ Ensure bed is completely dry before placing the Patient Return Electrode Pad on the OR table.

DO NOT

- ✗ Do not use hydrogen peroxide or hydrogen peroxide-containing materials to clean the Patient Return Electrode Pad.



- ✗ Do not use disinfectant containing more than 70% alcohol to clean the Patient Return Electrode Pad.



Please refer always to the Instructions for Use / Package Insert that come with the device for the most current and complete instructions.

Ethicon Endo-Surgery (Europe) GmbH
Hummelsbütteler Steindamm 71
22851 Norderstedt, Germany